

# **ATTENTION**

- The Hybrid Power Rod Cutter, PI-2477, is designed to **ONLY** cut Titanium or Cobalt Chromium rods of circular diameters 5.5mm, 6.0mm, and 6.35mm.
- Do **NOT** cut other shapes, sizes, or materials.
- Ensure rods are **ONLY** cut in the hole of matching diameter size.
- To prevent this cutter from seizing, ensure all moving parts, particularly the jaw components, are lubricated with water-soluble instrument lubricant prior to each use (e.g., Steris Hinge- Free® Instrument Lubricant or Neodisher® IP Spray).
- Operating this cutter requires a moderate-high amount of strength; ensure user is physically able to exert enough force for proper operation.
- Misuse may result in failure and is easily detectable.
- See IFU-013 for additional information.

**P·A·C·I·F·I·C**®  
INSTRUMENTS